


## A Simple Technique For Denture IDs

Using a Brother P-touch Labeler (pt2430pc) I print the patient's name on a ½ inch clear laminated label.

The 5 point font is Arial bold and it is set to print mirrored.




I've made a mold using .20 baseplate material and silicone putty in which I place some of my freshly mixed denture acrylic and secure with a rubber band.

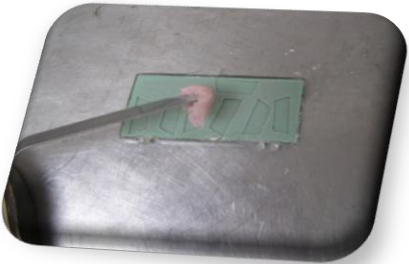

Microwave for 30 seconds. This is enough to stiffen the material yet still remain flexible. Trim with scissors. I have the trapezoid pattern so that it is easier to orient the name when placing in the flask (long edge is the top).



Place attached label in the flask in the desired location and cover with acrylic and press....trim...press...trim....press....trim...place in compress and cure.

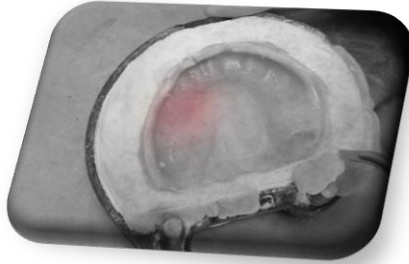

Deflask and finish in usual manner.

